

WomenONLY
FEMALE DESIRE.




SMARTBALLS

Product INFORMATION

EXPERT OPINION

Dr. Verena Breitenbach, Gynecologist in Baden-Württemberg, Germany about SMARTBALLS:

„The pelvic floor is a weak spot for most women. Unfortunately, in contrast to other muscles, it is almost never trained. Most women don't know where it is and cannot contract it knowingly. If the pelvic floor is weak, it can lead to bladder incontinence, bowel incontinence, lower abdominal pain and to a prolapse of the uterus. With SMARTBALLS women can effectively train the pelvic floor and strengthen it while improving their sex lives.“

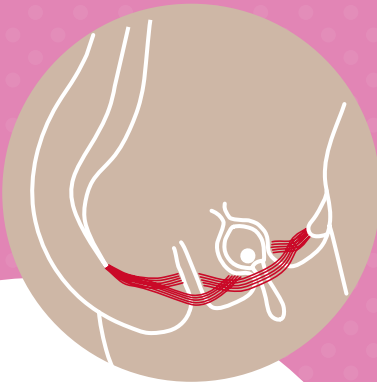

FEATURES

- Recommended by midwives and gynaecologists
- To strengthen pelvic floor muscles
- To prevent incontinence
- Tapered ball eases insertion
- Finger mould for “easy-in“ insertion
- Steady, intense rolling movement of balls
- Velvety soft silicone sheathing
- Flexible retrieval band of silicone, barely felt
- Almost completely silent
- Gentle to the body, hypo-allergenic
- Velvety soft silicone
- Quality “Made in Germany”

DEFINITION

SMARTBALLS make it possible to train the pelvic floor muscles in a playful way. The pelvic floor holds the lower part of the uterus as well as the constrictor muscle of the bladder and anus. A weakening of the pelvic floor muscles can be caused by connective tissue injuries, childbirth, chronic physical stress, poor posture or a pelvic operation. This then causes stress incontinence – or urinating when carrying heavy objects, sneezing, coughing, laughing, jumping etc.. With concentrated training it is possible to fix incontinence or prevent it all together. Plus a more fulfilling sex life is possible with strong pelvic muscles, because a woman has more control over the inner musculature:

- The woman has greater feeling during sex and ...
- the man has a better sensation when he is held more snugly.

CARE

- With the FUN FACTORY CLEANER, specially made for silicone toys and the genital areas.
- ... or with water and mild soap.

TRAINING

Die **SMARTBALLS** are inserted in the vagina with **TOYFLUID**. The retrieval band remains outside the body. The "easy-in" mould makes it easier to insert.

To set the inner balls of the **SMARTBALLS** in motion and to change the point of gravity of the balls everyday activities suffice:

- walking,
- climbing stairs, etc.

The vibration of the inner balls gives the pelvic muscles the feeling that it is about to lose hold of something (so-called "bio feedback"). The musculature reacts reflexively with contractions. The training then occurs automatically. After the training, **SMARTBALLS** can be easily removed using the retrieval band.

The **SMARTBALLS** can be used for several minutes or for several hours – whatever feels right. We recommend increasing the time little by little. To reach an optimal strengthening of the pelvic floor muscles the **SMARTBALLS** should be used regularly. For acute pelvic muscle weakness a daily use of ca. 30-40 minutes during normal daily activities should be enough.



SHAPE



SMARTBALL UNO

is ideal for women:



- ... for whom **SMARTBALLS DUO** are too big and would rather start with inserting just one ball.



- ... with a dropped uterus (for example after multiple pregnancies).
- ... with a tipped back uterus.
- ... who have had a hysterectomy.
- ... that want to begin their training start with smaller weights.



SMARTBALLS DUO

are ideal for women:



- ... who have already successfully trained with **SMARTBALL UNO** and would like to upgrade.



- ... without a dropped uterus (for example after multiple pregnancies).



- ... who do not have a tipped back uterus.



- ... who have not had a hysterectomy.



- ... that want to boost their workout and train with double weights.



TECHNICAL DATA

Material	100 % medical grade silicone (outside material), ABS plastic (ball(s) inside)
Sizes	4,5 cm, Ø 3,6 cm 1.8 in, Ø 1.4 in (SMARTBALL UNO) 10 cm, Ø 3,2 cm + Ø 3,6 cm 4.0 in, Ø 1.3 in + Ø 1.4 in (SMARTBALLS DUO)
Weight	36 g 0.08 lb (SMARTBALL UNO) 72 g 0.16 lb (SMARTBALLS DUO)

SALES TIPS

SMARTBALLS are NOT considered a sex toy. Yet after intensive or regular training the strengthened pelvic muscles can lead to more fun during sex.

SMARTBALLS are a mobile training equipment for the pelvic muscles. For more fun, more control, and better sex.

"Control your muscles and control HIM during sex."

Let the customers feel the velvety soft and smooth silicone surface. Trusted **FUN FACTORY** quality – Made in Germany!

SMARTBALLS – recommended by midwives and doctors.

The optimized shape of the **SMARTBALLS** (tapered at the end) and the improved rolling movements of the balls can be felt and heard. Let the customer shake the **SMARTBALLS**. The balls are very quiet, pleasantly smooth and particularly elegant.

SMARTBALLS by **FUN FACTORY** ... often imitated, never duplicated.

An independent jury named Action Plagiarius, confirmed that many have unsuccessfully tried to imitate.

We are the **ORIGINAL**, awarded with several design prizes!